BDD07

Use only with PharmEvo
DPI Capsules

Description:

The AirCare device is a single dose inhalation device used to inhale the dry powder contained in PharmEvo dry powder inhaler capsules.

For administration of the following PharmEvo dry powder inhaler products:

- ▶ Budesonide (Budesonide + Formoterol Fumarate Dihydrate) – 200/6mcg, 400/12mcg
- ▶ TioCap (Tiotropium) – 18mcg
- ▶ Forsonide (Salmeterol + Fluticasone Propionate) – 50/250mcg, 50/500mcg

Warning & Precautions:

It is important for patients to understand how to correctly administer PharmEvo dry powder inhaler capsules using the AirCare device. Please refer to detailed instructions given in this leaflet.

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Do not use the device when it is wet. You may clean the outside of the mouthpiece with a clean damp cloth, periodically rinse with water.

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PharmEvo dry powder inhaler capsules should only be administered via the AirCare device. AirCare device should not be used for administering other medications.

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The contents of PharmEvo dry powder inhaler capsules are for oral inhalation only and must not be swallowed.

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Do not store PharmEvo dry powder inhaler capsules in the AirCare device.

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Care must be taken not to allow the powder to enter into the eyes as this may cause blurring of vision and pupil dilation.

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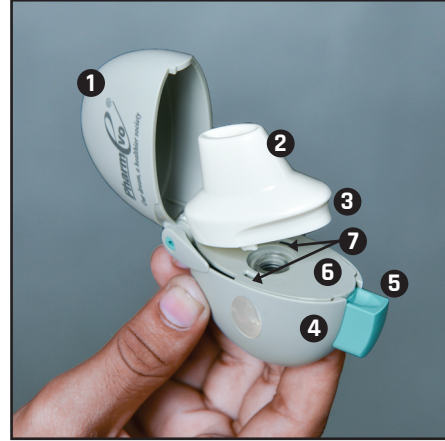
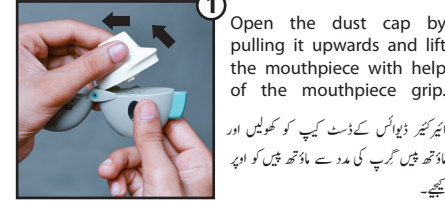
Instructions for Use

1 Open the dust cap by pulling it upwards and lift the mouthpiece with help of the mouthpiece grip.

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2 Remove the capsule from the blister right before use. Do not store the capsule in the AirCare device.

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3 Insert the capsule in the capsule chamber of the AirCare device.

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4 Close the mouthpiece until you hear a click, but leave the dust cap open.

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5 Hold the AirCare device with the mouthpiece upright. Press the piercing button once completely and release. This is how you make holes in the capsule so that you get medicine when you breathe in.

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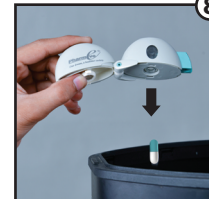
6 Breathe out completely. Do not breathe (exhale) into the mouthpiece of the AirCare device at any time.

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
7 Raise the AirCare device to your mouth and close your lips tightly around the mouthpiece. Keep your head in an upright position. Breathe in slowly and deeply so that you hear or feel the capsule vibrate. Breathe in as much as you can and hold your breath as long as possible. At the same time, remove the AirCare device from your mouth. Breathe normally again. Repeat steps 6 & 7, in order to empty the capsule completely.

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8 After taking your dose, open the mouthpiece. Tip out the used capsule and throw it away.

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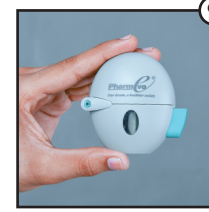9 Close the mouthpiece and dust cap before storing your AirCare device.

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10 Clean the AirCare device once a month. Open the dust cap and mouthpiece. Then open the base by lifting the piercing button. Rinse the AirCare device with warm water and check if any powder buildup or capsule fragments remain.

Dry the AirCare device thoroughly by tipping excess of water out on a paper towel and air-dry afterwards, leaving the dust cap, mouthpiece and base open. Do not use detergents or a dishwasher to clean the AirCare device. It takes 24 hours to air dry, so clean it immediately after use so that it will be ready for your next dose. Do not use a hair dryer to dry the AirCare device. Do not use the AirCare device when it is wet. If needed, you may clean the outside of the mouthpiece with a clean damp cloth.

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PharmEvo®
Our dream, a healthier society

www.pharmevo.biz

Plot # A-29, North Western Industrial Zone, Port Qasim, Karachi-75020, Pakistan. Establishment License No.: ELI-00055

Manufactured by:
Shanghai Huarui Aerosol Co., Ltd.
No. 222, Yuanchun Road, Pudong New District, Shanghai, 201399, P.R. China.

AIRCARE® & PharmEvo®
are registered trademarks of PharmEvo (Pvt.) Ltd.